


## Guidelines Based on Chest/Bust Measurement

### Measurement Instructions

Have the patient remove any outerwear like a sweater or jacket and have them stand straight with arms at their side. The patient should *not* inhale to overly expand their chest, but be in a natural state. Using a tailor's or sewing tape, take the chest / bust measurement under the arms and across the **largest** part of the chest. Pull the measuring tape around without overly stretching or tightening the tape, a relaxed measurement. Take the measurement with the patient's arms at their side (not with their arms up or outwards).

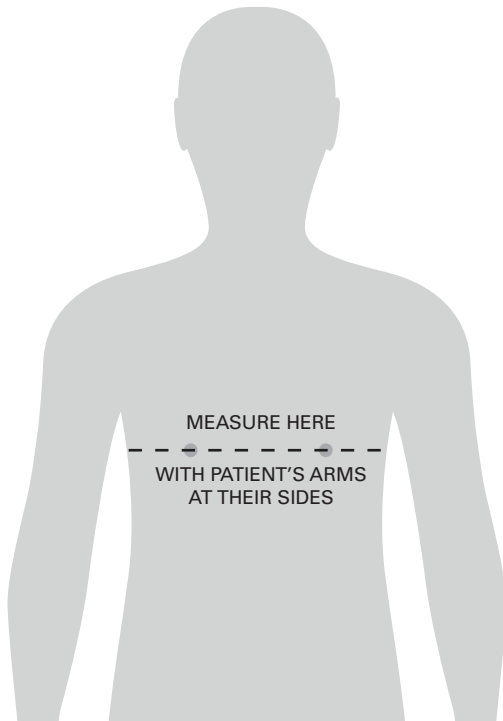

### SIZING CHART

AffloVest Size	Chest/Bust Measurement
Extra Small (XS)	30 – 32 inches
Small (S)	33 – 38 inches
Medium (M)	39 – 44 inches
Large (L)	45 – 52 inches
Extra Large (XL)	53 – 60 inches

#### Upon receipt of AffloVest, please note:

1. A 1 inch overlap of the vertical yellow seams on the front of the vest is acceptable for all sizes
2. A gap between the vertical yellow seams is also acceptable and can be up to 5 inches on XS, S and M sizes and up to 8 – 10 inches on L and XL sizes.

#### Please note:

- Fit and sizing vary from person to person.
- The fit of the chest is more important than the fit around the waist.
- Using the front adjustment straps, the AffloVest should fit very snugly on the chest, but not limit the patient's ability to take in a full, deep breath.
- If needed, adjust the shoulder size snaps so that the AffloVest is positioned high enough on the chest to ensure effective treatment.